Equine LTS Supreme

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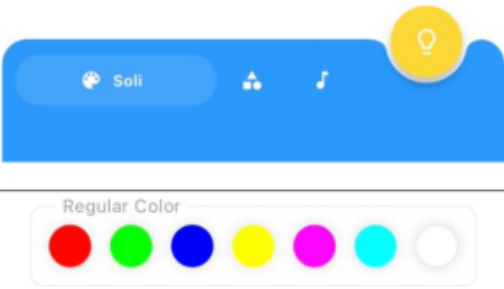
	<p>Tap on the 💡 to choose between the different colors/frequencies.</p> <p>The button in the middle you will get to a menu where you can do different programs, which gives the opportunity to use pulsing,</p> <p>Here you can control the color/frequencies of your Supreme.</p> <ul style="list-style-type: none">❤️: Your Supreme will only use infrared.♥️: Your Supreme will only use red.♥️: Your Supreme will only use blue.♥️: Your Supreme will only use red.♥️: Your Supreme will only use blue♥️: Your Supreme will use all three frequencies (blue, red and infrared) at the same time.
---	---

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	<p>11 Red comet. Program with pulsing effect of infrared light 12 Green comet. Program with pulsing effect of red light. 13 Blue comet. Program with pulsing effect of blue light. 17 Whiter comet. Program with pulsing effect of both infrared, blue and red light at the same time.</p> <p>For all programs above, set the speed to 10 and the effect length to 25.</p> <p>If you tap on the ⓘ you can remove programs from the menu "Favorite". Tap on the ♥ when you are on the program that you want to save under "Favorite", so that it's always easy to find.</p>
--	--

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